


1

## Timing Cycles: Single Chamber


- There is one basic interval connecting 2 consecutive events, pacing or sensing: AA or VV.
- There are two basic transitions, or pacing events that cause the basic interval to be reset (reinitiated).
- If the interval times out, a stimulus is released, and the interval starts over.
- If a spontaneous event is sensed, the interval is reset without pacing, the pending stimulus being “inhibited.”

2

2

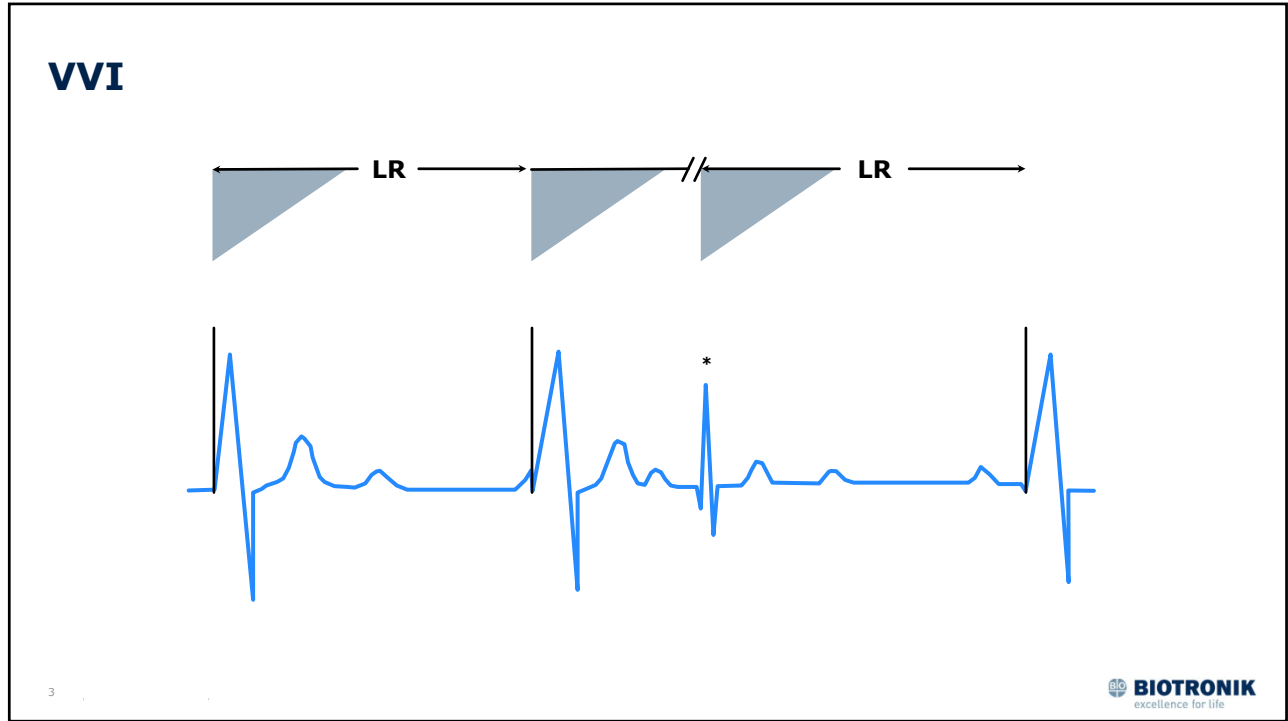

3

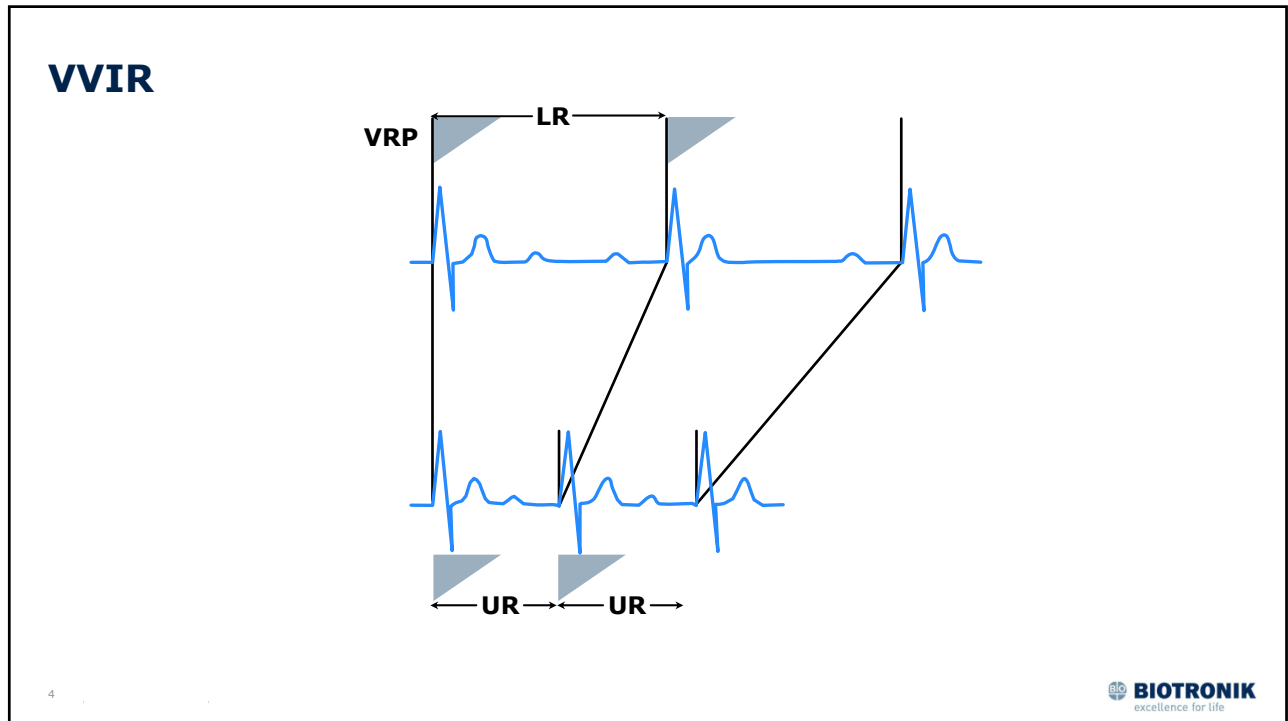

4

## Timing Cycles: Dual Chamber

- Two basic intervals: AV & VA; their sum is the cycle duration or “cycle length” AA or VV.
- The DDD mode has five basic transitions, each of which resets (re-initiates) either the AV or the VA interval depending upon the event that initiates the transition.

5

5

## Five Transitions of DDD Timing

### Event


AV interval times out

### Response

Pace V, begin VAI

6

6

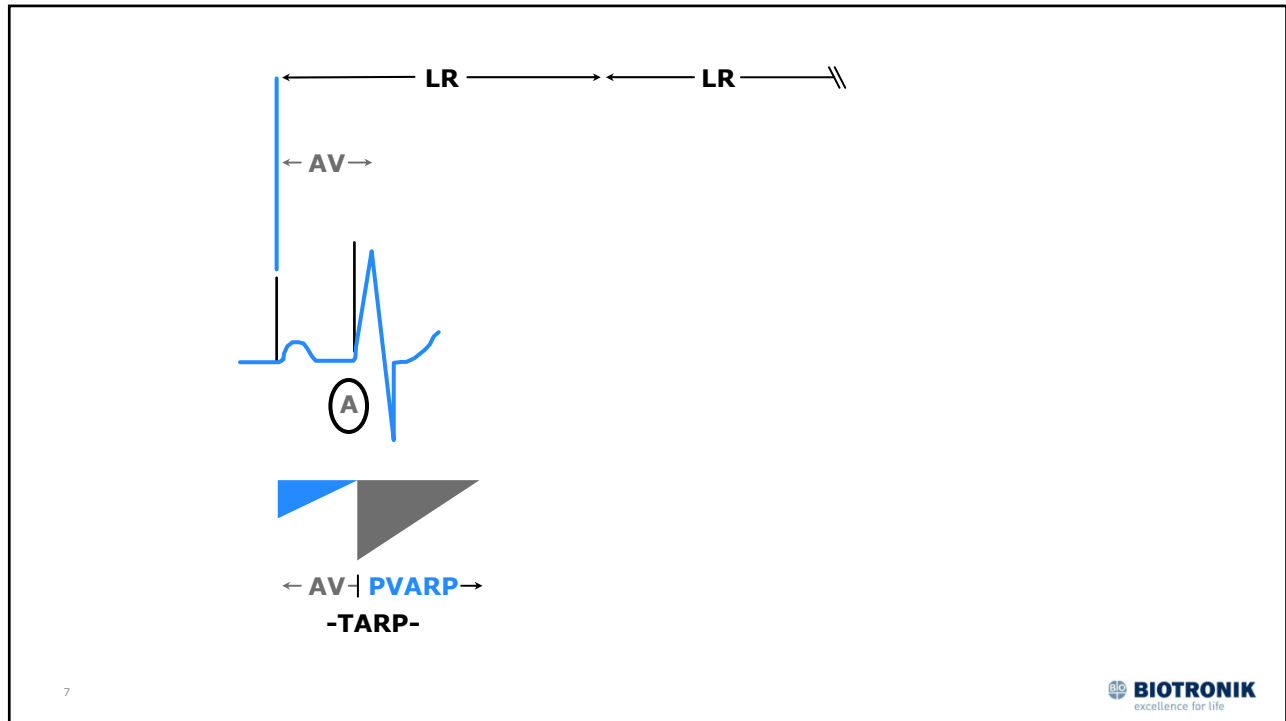

7

## Five Transitions of DDD Timing

### Event


AV interval times out  
VA interval times out

### Response

Pace V, begin VAI  
Pace A, begin AVI

8

8

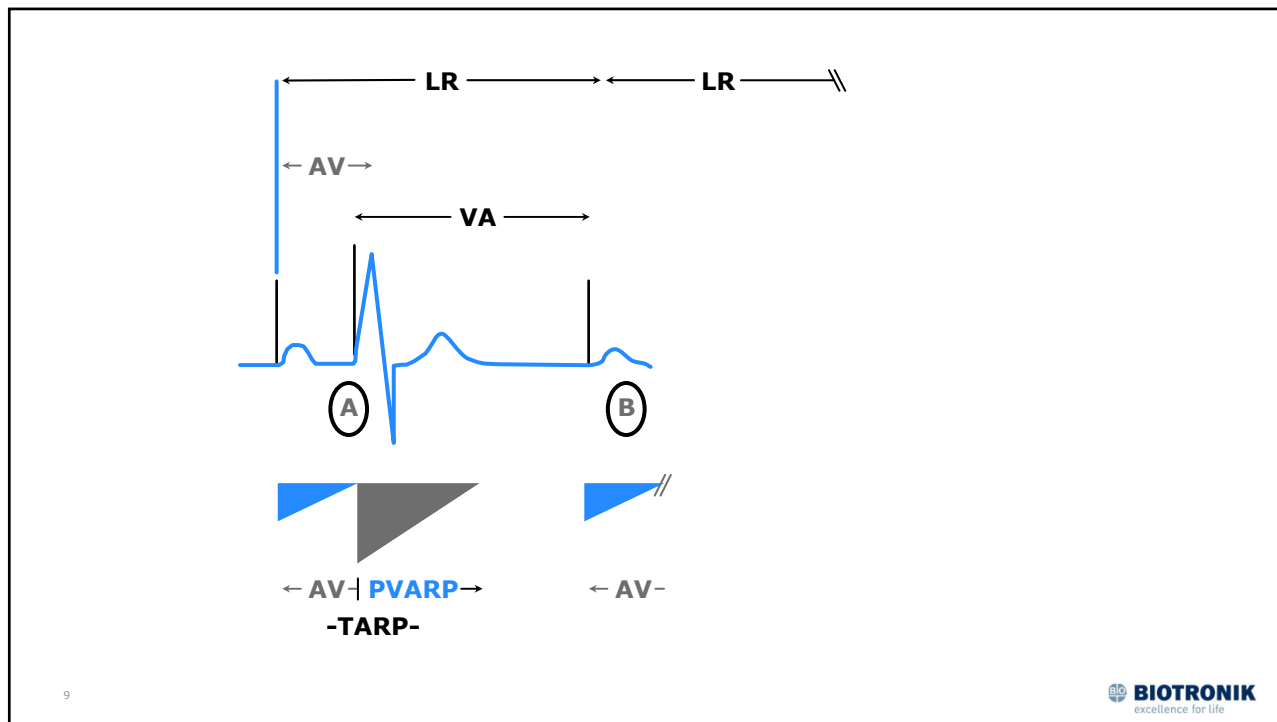

9

## Five Transitions of DDD Timing

### Event


- AV interval times out
- VA interval times out
- V sense during AVI
- V sense during VAI

### Response

- Pace V, begin VAI
- Pace A, begin AVI
- Begin VA/no pacing
- Re-start VA/no pacing

10

10

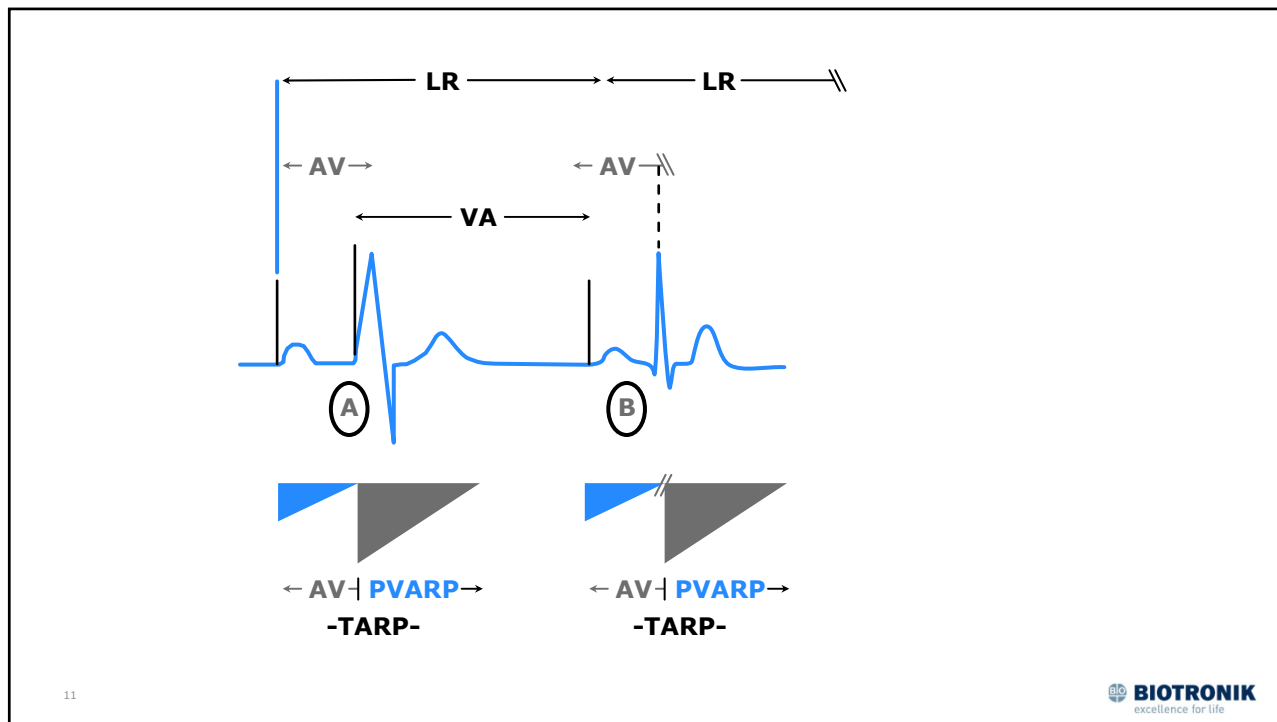

11

## Five Transitions of DDD Timing

### Event


AV interval times out  
 VA interval times out  
 V sense during AVI  
 V sense during VAI  
 A sense during VAI

### Response

Pace V, begin VAI  
 Pace A, begin AVI  
 Begin VA/no pacing  
 Re-start VA/no pacing  
 Begin AVI/no pacing

12

12

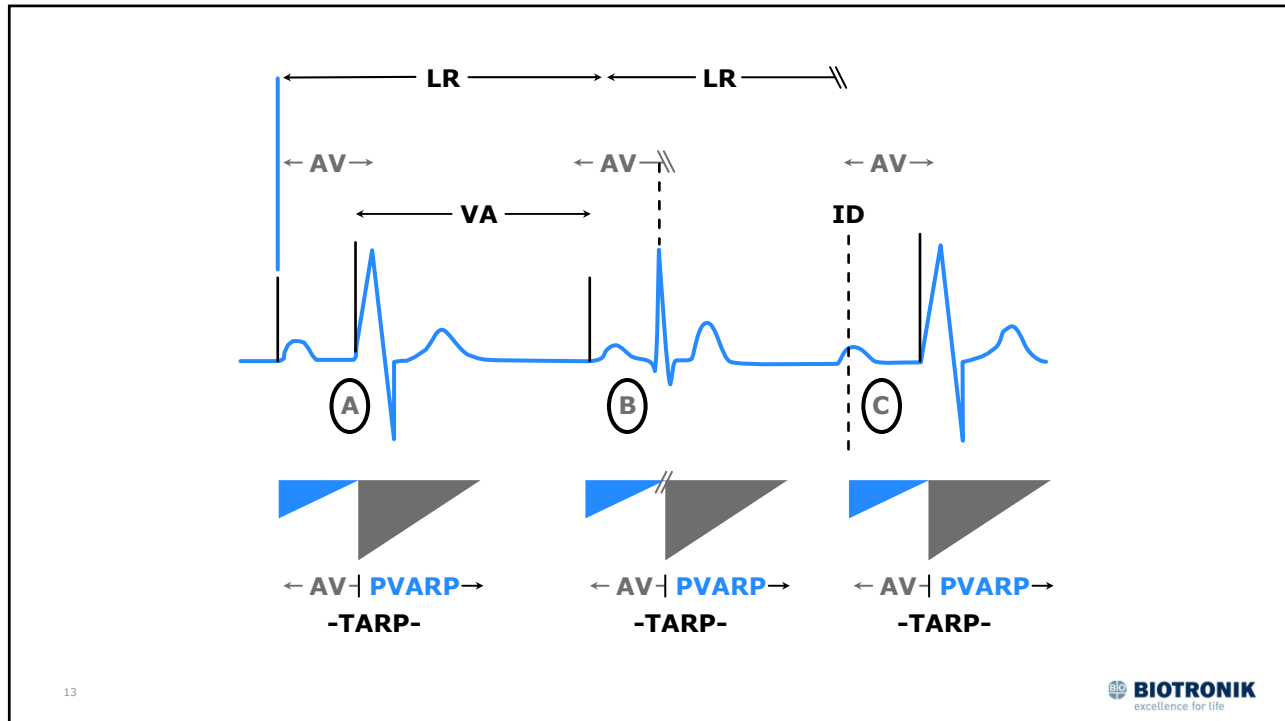

13

Which of the following defines the maximum tracking rate of a DDD pacemaker?

1. PVARP
2. VRP + AVI
3. TARP
4. AVI + Blanking period
5. TARP - VRP

14

14

## TARP = AVI + PVARP


- AVI = 200 ms
- PVARP = 400 ms
- TARP = 600 ms = 100 bpm

*TARP = Total Atrial Refractory Period*

*PVARP = Post-ventricular atrial refractory period*

15

15

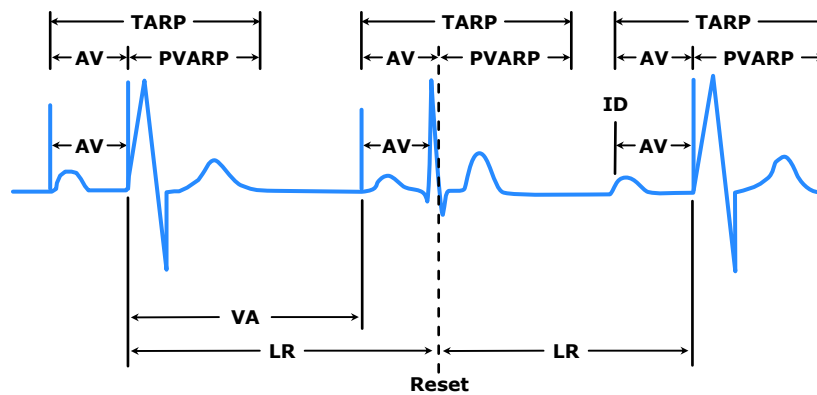

16

16



## Timing Cycles


- During a refractory period, sensed events are ignored for timing purposes – although they would usually be annotated on ‘marker’ channel.
- During a blanking period, the sensing amplifier is turned off and sensing cannot take place.
- The pattern of refractory and blanking periods plays a critical role in determining the beat-to-beat behavior of the pacemaker.

17

17

If the PM represented in this schematic is functioning normally, what is the pacing mode?

1. DDD
2. AAI
3. VVI
4. VDD
5. VAT

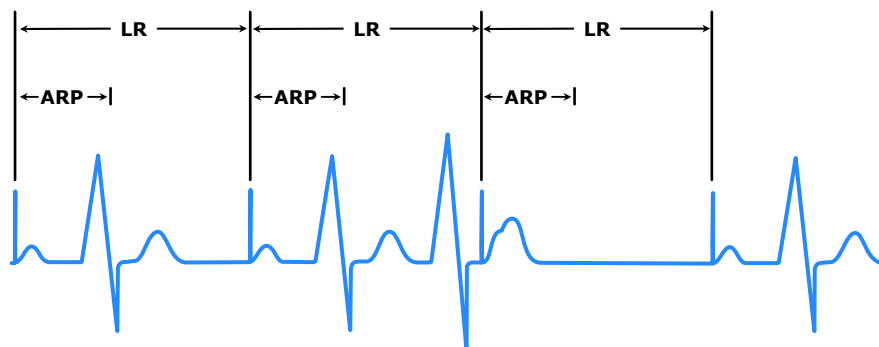


18

18

If the PM represented in this schematic is functioning normally, what is the pacing mode?

1. DDD
2. AAI
3. VVI
4. VDD
5. DDI

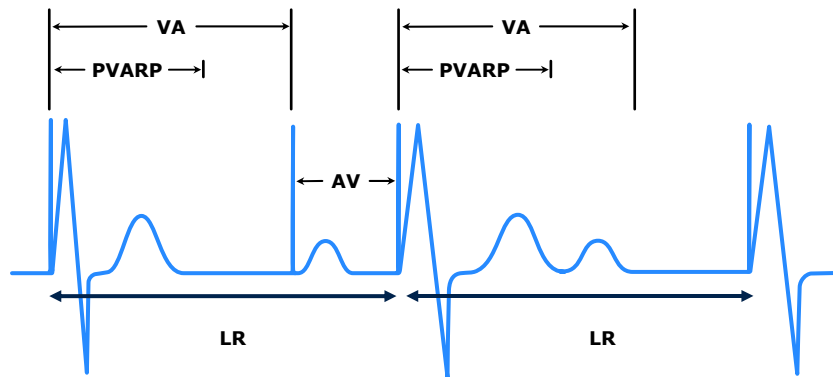


19

19

If the PM represented in this schematic is functioning normally, what is the pacing mode?

1. DDD
2. AAI
3. VVI
4. VDD
5. VAT

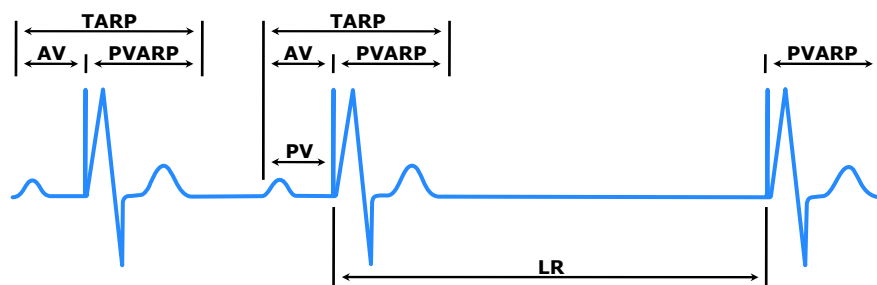


20

20

## The Timing Cycle Is Compatible With:

1. AAI with farfield sensing
2. VDD pacing mode
3. Normal blanking
4. Normal VVI

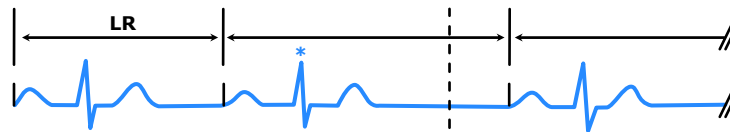


21

21

## The Timing Cycle Is Compatible With:

1. **AAI with farfield sensing**
2. VDD pacing mode
3. Normal blanking
4. Normal VVI

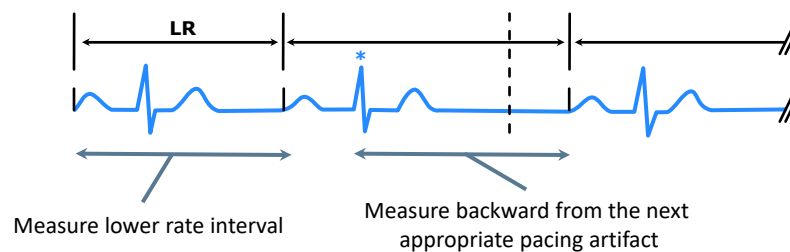

22

22

### Specific Timing Cycles

- AAI or AAIR and appropriate lack of response to ventricular events
- VDD or VDDR and reversion to VVI timing in the absence of atrial activity
- DDI or DDIR and the lack of atrial tracking (*in DDI--lower rate = upper rate*)

23


23

### Timing Cycles: Crosstalk


- Unwanted detection in one channel of a signal from another channel
- Most common: afterpotential from atrial output sensed by V channel and resets VA interval
- AV crosstalk easier to avoid with bipolar sensing configuration, less sensitive to far-field signals

24

24

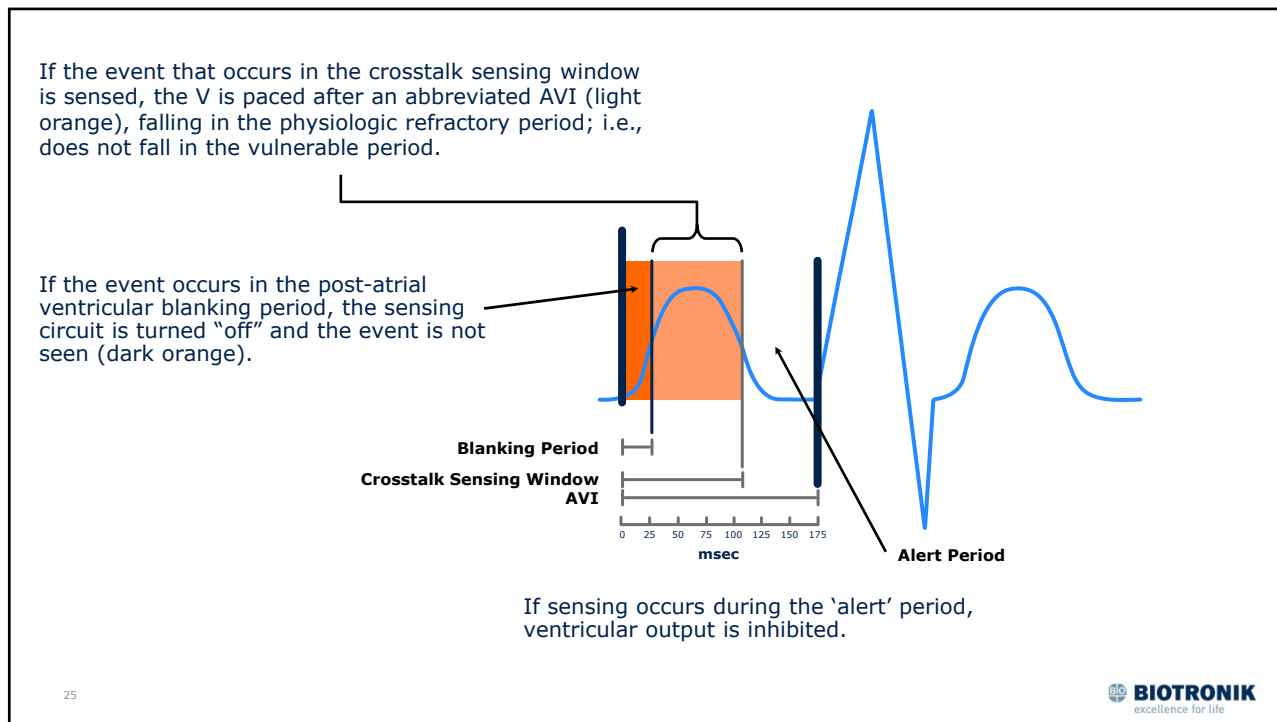

25

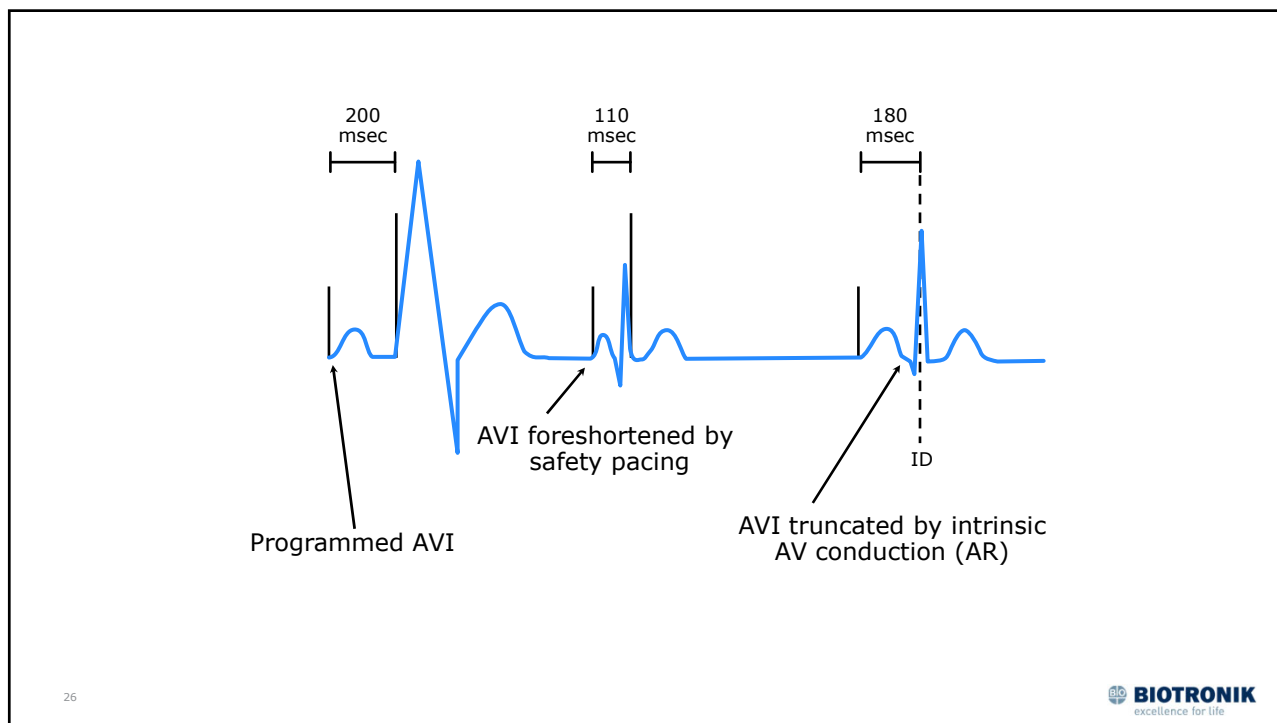

26

## Timing Cycles: Crosstalk

AV crosstalk may be eliminated by:


- Increasing the blanking period (*so that ventricular sensing resumes later*)
- Decreasing the atrial stimulus voltage or duration (*if adequate safety margin can be maintained*)
- Making ventricular channel less sensitive

27

27

**Programmed AV = 220; The labeled QRS complex (\*) occurs in the:**

1. Crosstalk sensing window
2. Alert portion of AVI
3. Post-atrial ventricular blanking period

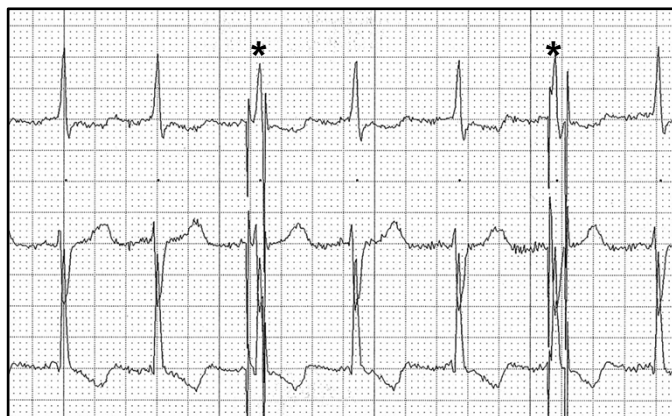


28

28

**Programmed AV = 220; The labeled QRS complex (\*) occurs in the:**

1. Crosstalk sensing window
2. Alert portion of AVI
3. Post-atrial ventricular blanking period

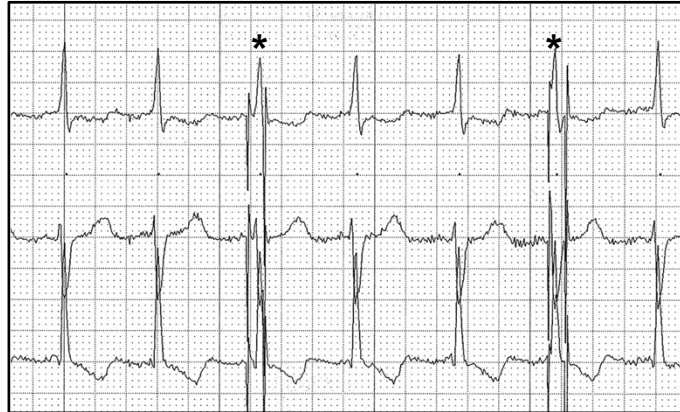

29

 **BIOTRONIK**  
excellence for life

29

## Upper Rate Behavior


Components of basic timing

- TARP (AVI + PVARP)
- Dynamic AVI
- Rate-adaptive parameters
- Timing system


30

 **BIOTRONIK**  
excellence for life

30

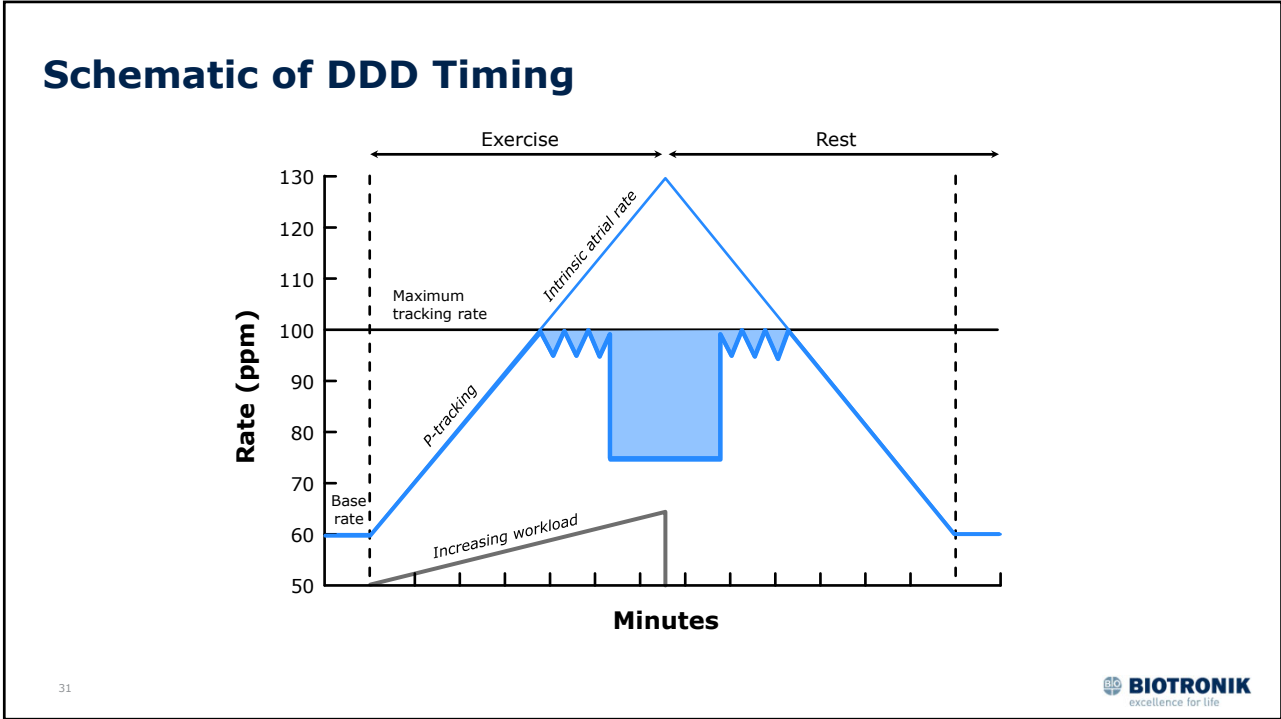


31

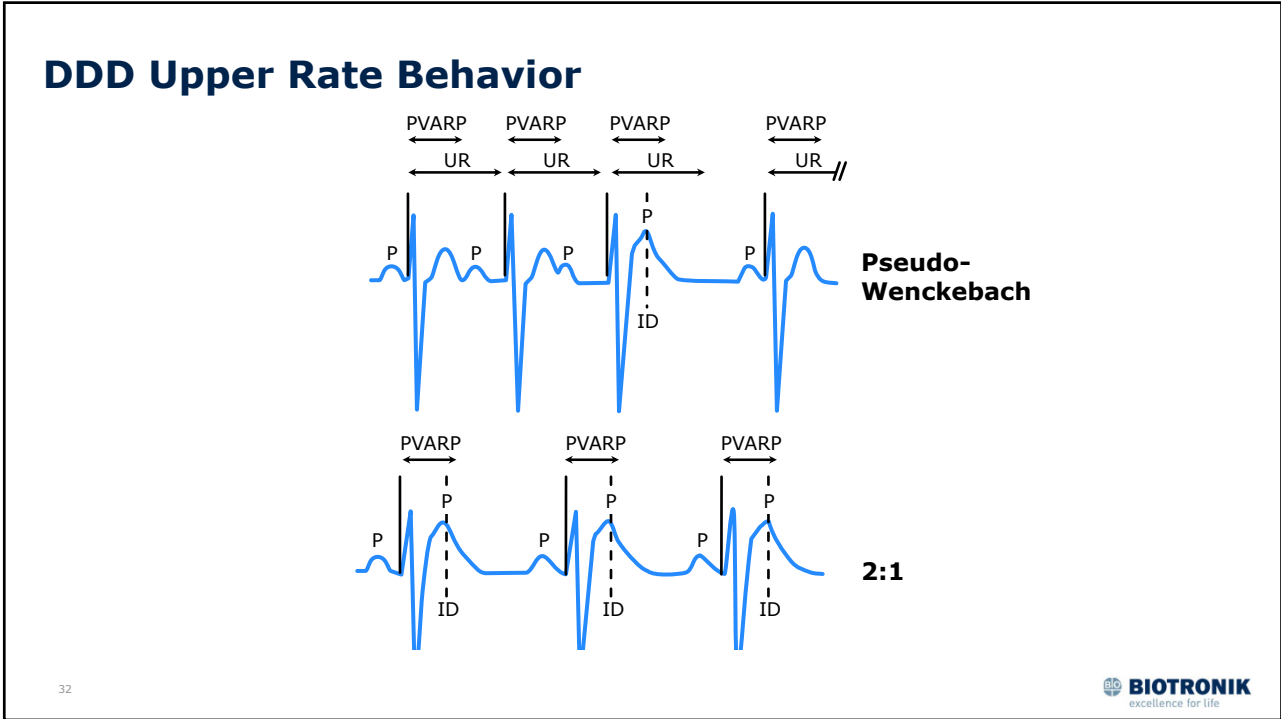

32



### Schematic of DDD Upper Rate Behavior

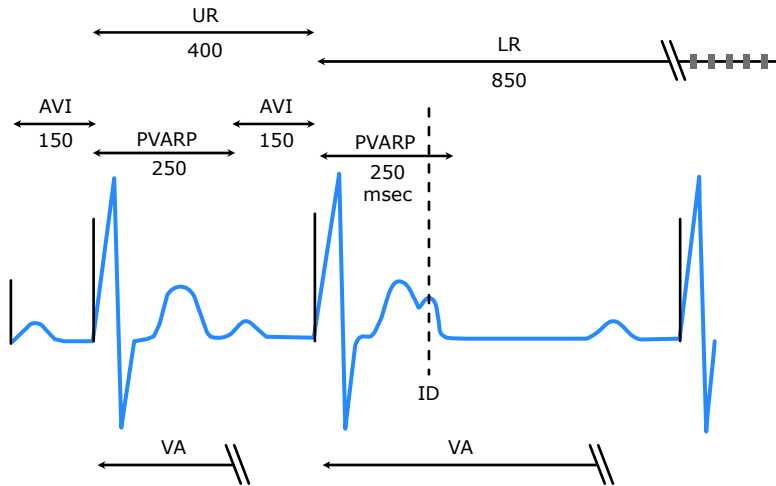


33



33

### Schematic of DDD Timing

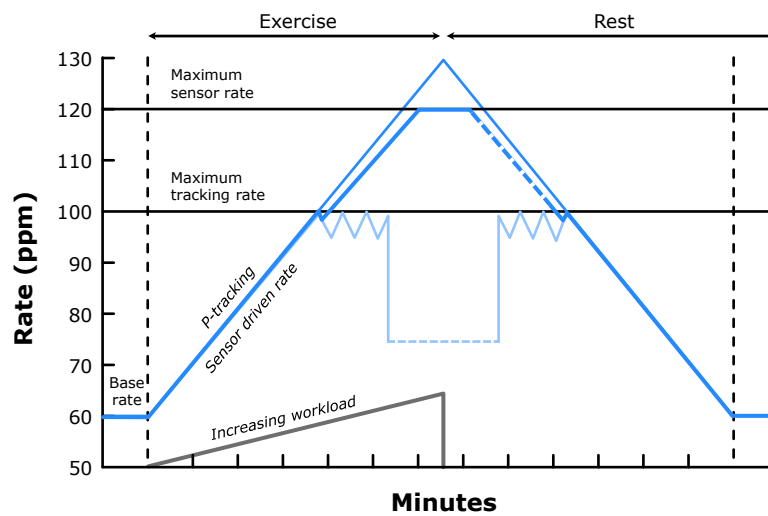


34



34


### Schematic of Sensor-Driven Rate Smoothing

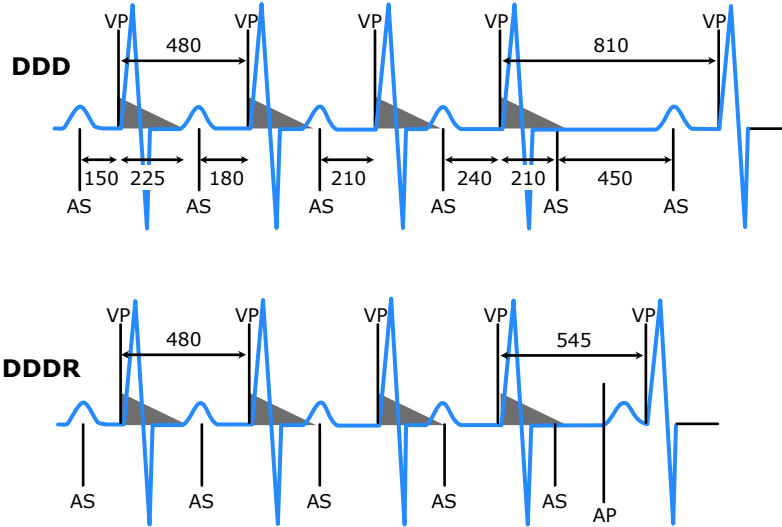

35




35

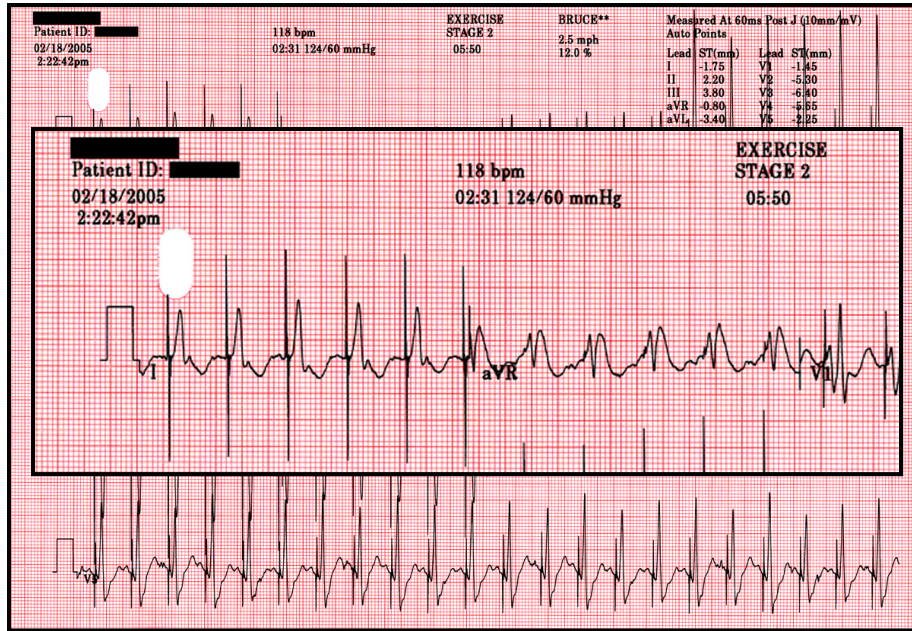


36



36

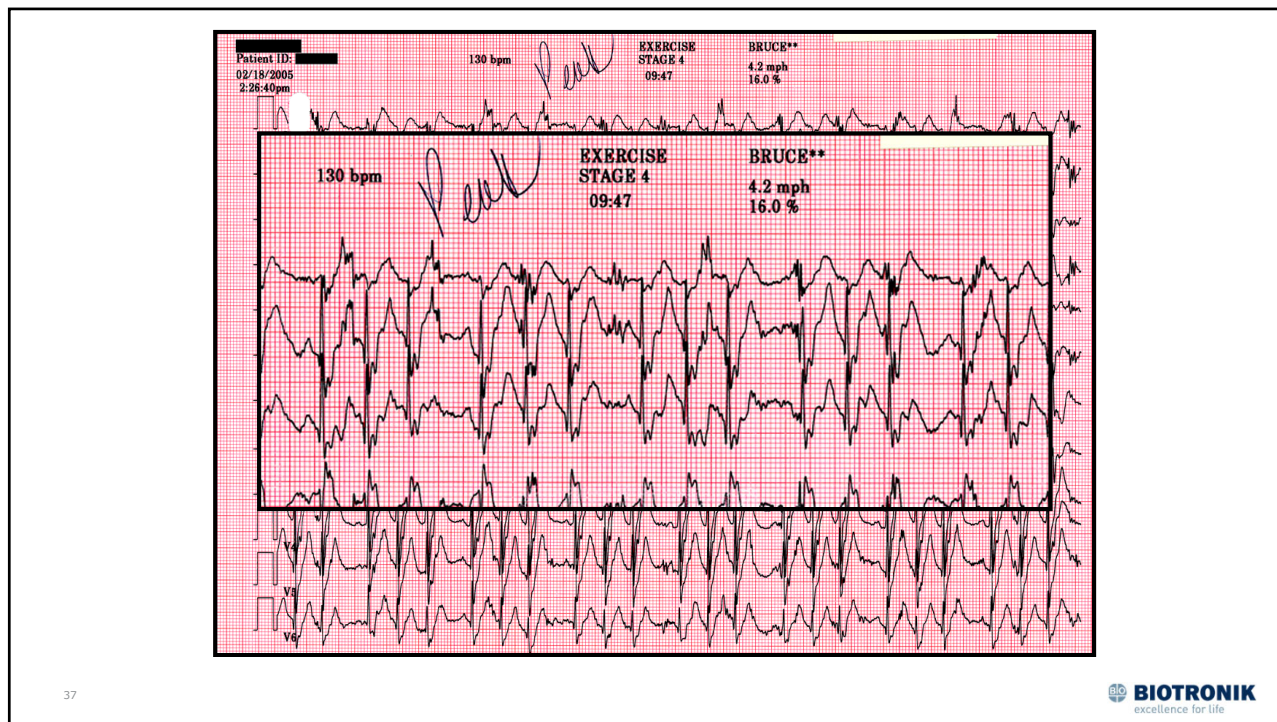

37

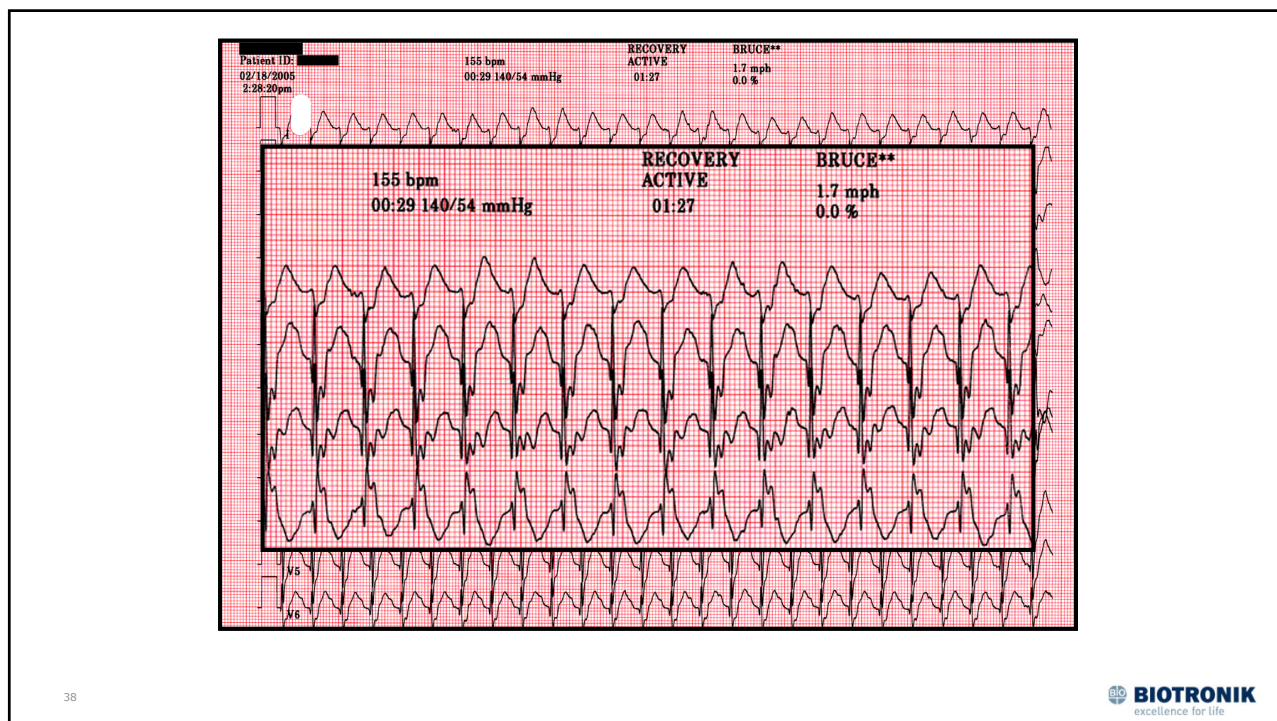

38

## Timing Cycles

- Understanding basic timing cycles is critical for ECG interpretation of all CIEDs.
- Exceptions and nuances exist for every manufacturer.
- If interpretation doesn't fit basic timing cycle principles, contact technical services for assistance.